



# INSTRUCTIONS OF USE

## PolySpine Profile Tool

**Step 1.** Clip together the individual pieces of the PolySpine Profile Tool until the required length is achieved. Ensure each tooth is positioned at number '4' before clipping in.

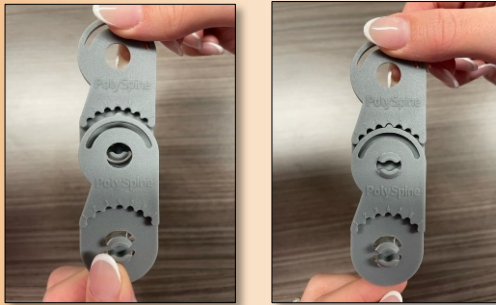

**Step 2.** Place the flat edge of the device against the body part being recorded and carefully shape the tool to the individual's profile.

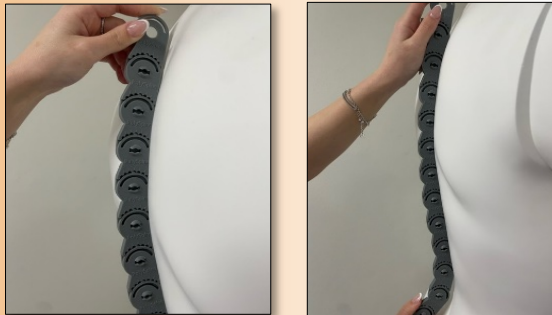

**Step 3.** Transfer the PolySpine Profile Tool to the intended piece of equipment and shape to suit.

**Step 4.** Record the Unique Profile Number (UPN) by simply reading the numbers from top to bottom.



*Example UPN: 4344444*